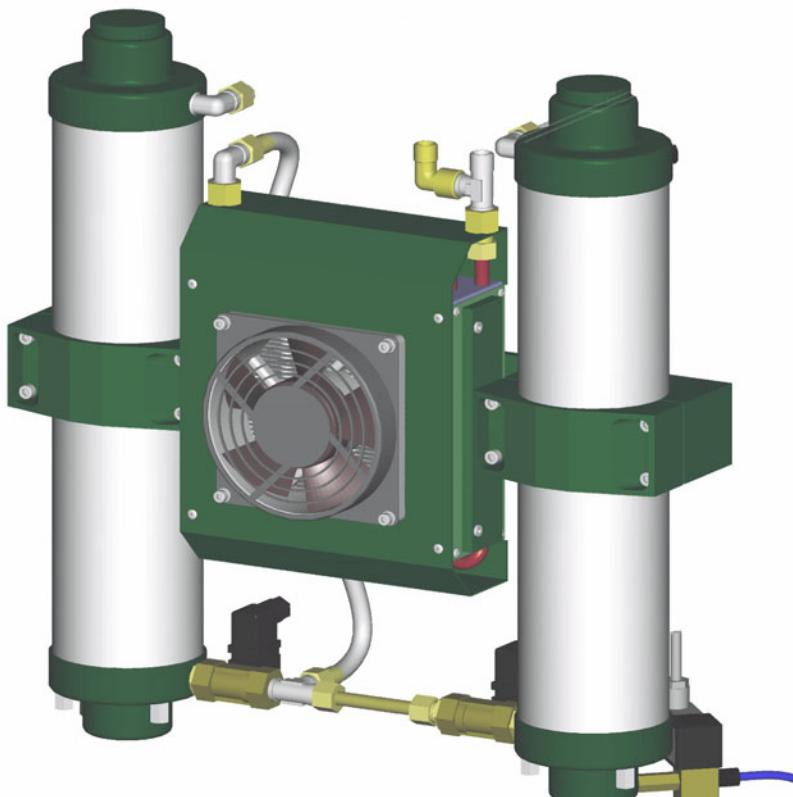
**ekom**

# Monzun M2 /M2.1 Monzun M3 /M3.1

SUŠIČ VZDUCHU ADSORBČNÝ

## NÁVOD NA POUŽITIE

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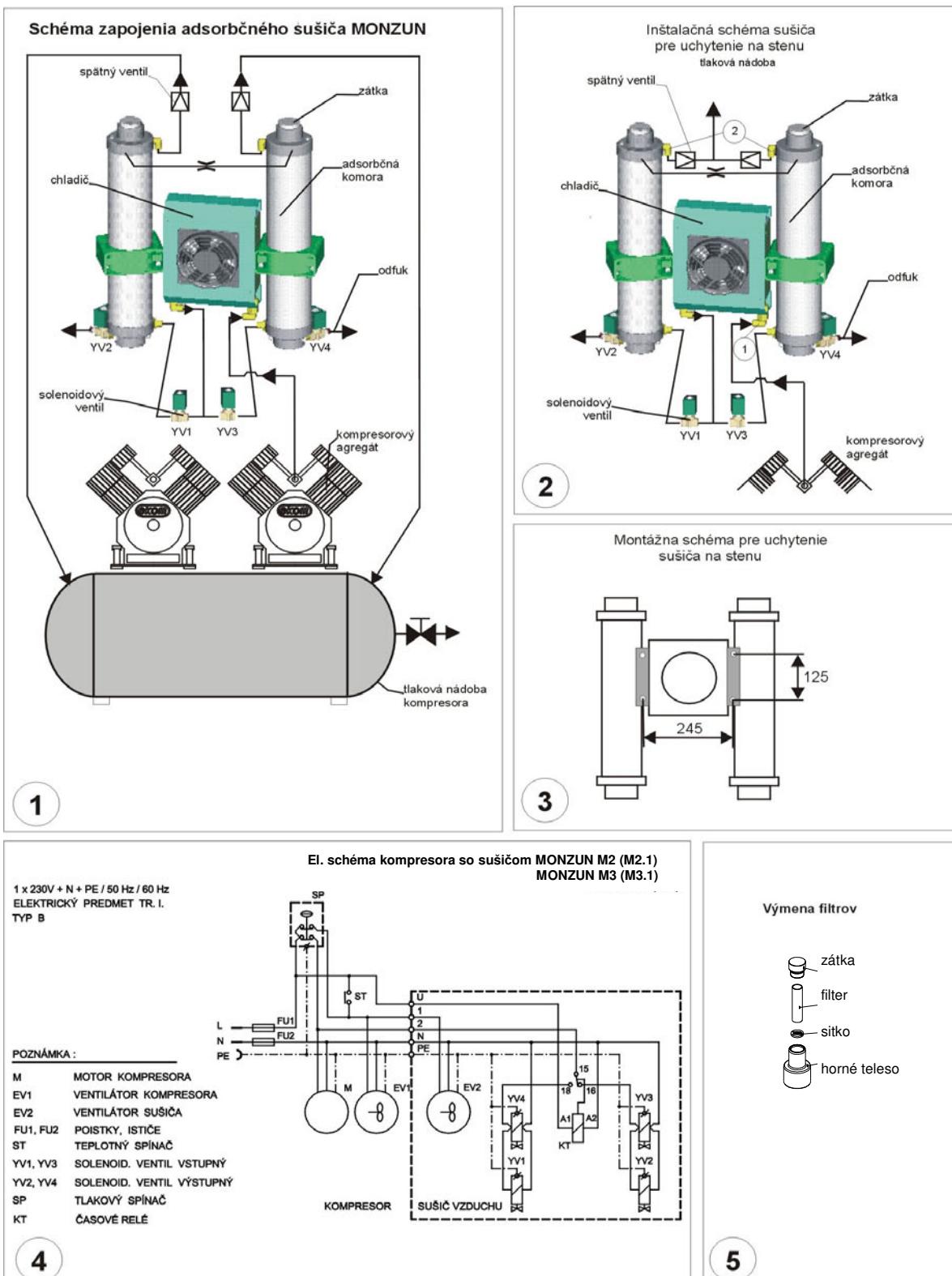
## SCHÉMA ZAPOJENIA

**CONTENTS**

|  |          |
|--|----------|
| <b>PURPOSE AND USE .....</b>                 | <b>6</b> |
| <b>PRODUCT DESCRIPTION AND FUNCTION.....</b> | <b>6</b> |
| <b>TECHNICAL SPECIFICATION.....</b>          | <b>6</b> |
| <b>BASIC OUTFIT .....</b>                    | <b>6</b> |
| <b>ASSEMBLY AND INSTALLATION .....</b>       | <b>6</b> |
| <b>OPERATION OF PRODUCT.....</b>             | <b>7</b> |
| <b>MAINTENANCE OF PRODUCT .....</b>          | <b>7</b> |
| <b>TROUBLESHOOTING AND REPAIRS .....</b>     | <b>7</b> |
| <b>CONNECTION LAYOUT .....</b>               | <b>8</b> |

## PURPOSE AND USE

Adsorption driers MONZUN M2/M2.1 and MONZUN M3/M3.1 are determined for drying of compressed air of oil-free compressors used for driving dental equipment in dental consulting rooms and laboratories. Described equipment could be used in another areas of health care, in laboratories, food industry and everywhere where a clean air is demanded without humidity and impurities and its parameters could be appropriate. The equipment is designed for inner room temperatures of air from +5°C to +35°C

## PRODUCT DESCRIPTION AND FUNCTION

Adsorption drier MONZUN M2 is proposed for the fixation to EKOM compressors of the relevant type. Driers MONZUN M2.1 and MONZUN M3.1 are adapted for fixation onto wall and the connection to oil-free compressors EKOM, but also to compressors from another manufacturers.

Adsorption driers MONZUN M2 / M2.1 and MONZUN M3 / M3.1 work in automatic mode of drying and regeneration.

Drying and regeneration take place at the same time in two separate chambers. Firstly, cooling of compressed air is done in a cooler (Fig. 1.1) with forced air circulation and then it is dried and filtrated in first chamber. Meanwhile, it comes to the regeneration of drying substance inside the second chamber by dried air taken from first chamber and to the exclusion of entrapped water outside the chamber..

After elapsing preset time, the mode of chambers is changed and process of drying and regeneration are executed in chambers in reverse order.

## TECHNICAL SPECIFICATION

### MONZUN M2 / M2.1

### MONZUN M3 /M3.1

|  |                              |
|--|------------------------------|
| • Nominal voltage / frequency:                 | 230 V ± 10%, 50 - 60 Hz      |
| • Power input:                                 | 52 VA                        |
| • Amount of regenerated air at pressure 5 bar: | do 280 lit.min <sup>-1</sup> |
| • max. working pressure:                       | 8 bar                        |
| • Degree of drying:                            | atmospheric dew point -20 °C |
| • Weight:                                      | 20 kg / 21 kg                |
| • Dimension in mm (š x h x v): Mx              | 520 x 200 x 540              |
| Mx.1   | 520 x 230 x 580              |
|  | do 140 lit.min <sup>-1</sup> |
|  | 18 kg / 19 kg                |
|  | 440 x 175 x 510              |
|  | 440 x 230 x 510              |

## BASIC OUTFIT MONZUN M2 / M2.1 M3 / M3.1

|   |          |
|---|----------|
| • Pressure hose 2 x DN8x600 or 1 x DN8x1200         | 1 ks (*) |
| • Pressure hose DN8x1600                            | 1 ks (*) |
| • Screw 5x50  | 4 ks (*) |
| • Collection vessel for condensate                  | 1 ks     |
| • PA hose for condensate outlet                     | 2 ks     |
| • Filter – modification 4KB-650.....024000653-000.. | 2 ks     |
| • Dowel pin 10                                      | 4 ks (*) |
| • Nozzle (see pg.3) – only without compressor       |          |

(\* )valid only for MONZUN M2.1/ M3.1

## ASSEMBLY AND INSTALLATION

Air driers **MONZUN M2**, **MONZUN M3** are assembled at manufacturer's place directly to the relevant type of compressor.

Installation of these types of driers onto compressors, that are already in operation, is to be made according to separate installation manuals delivered together with the product.

### WARNING !

 Prior starting the work disconnect the appliance from electric mains and reduce pressure in air chamber to zero!

When installing driers **MONZUN M2.1**, **MONZUN M3.1** it is necessary to drill holes in the wall at the proximity of compressor according to figure (see Fig. 1.3), to fix dowel pins and attach drier using screws.

Connect pressure hose of air from output of compressor aggregate to the input of cooler (position 1). Interconnect mutual output of drier (position 2) via pressure hose with air chamber.

Prior connecting drier to air chamber, that was used with compressor without drier, it is necessary to clean interior surface of air chamber and perfectly remove condensed liquid.

Then interconnect electric part of drier with compressor according to wiring diagram in accord with valid regional regulations.

Prior connecting drier to air chamber, that was used with compressor without drier, it is necessary to clean interior surface of air chamber and perfectly remove condensed liquid.

Then interconnect electric part of drier with compressor according to wiring diagram in accord with valid regional regulations.

## OPERATION OF PRODUCT

Appliance works together with compressor automatically and it requires no attendance for its operation. It is not necessary to sludge pressure vessel since pressure air in the vessel is already dried

## MAINTENANCE OF PRODUCT

**Prior intervention to the appliance it is necessary to reduce air pressure in air chamber to zero and disconnect the appliance from electric mains.**

The appliance is construed and manufactured so that its maintenance was minimal. At regular operation, it is necessary to replace filters in the upper parts of adsorption chambers once a year so as after repair of dysfunction which could pollute them.

After unscrewing the body plug, remove filtration pad. Then clean the screen if necessary. It is possible to check or to change the filling (when dirty or to dusty or the dryer doesn't dry). Put a new filter. Refit dryer plug to the dryer head and tighten it by hand (clockwise).

## TROUBLESHOOTING AND REPAIRS

**Prior starting the work disconnect the appliance from electric mains and reduce pressure of air in air chamber to zero!**

Activities regarding remedy of failures can be executed only by trained employee of service organization

Drying unit does not dry

- non-functioning solenoid valve - repair or replace the valve
- clogged nozzle of regeneration air - clean or replace the nozzle \*
- non-functioning ventilator of cooler - replace ventilator
- repair or replace the valve - check the adjustment of timer or replace it
- white liquid escapes from chambers - replace drying substance and lower board filter, re-seal and check tightness, rub O-ring with soap water

After the remedy of failure and reverse assembly of drier, it is necessary to perform the regeneration of drier, best at continuous operation of compressor at pressure of 0.6-0.7 MPa for at least 1 hour and then to check air drying.

\*) Size of regeneration nozzle:

M3, M3.1

Up to 70 l/min (DK50 PLUS)  
Up to 140 l/min (DK50 2V)

ø0,5mm  
ø0,7mm

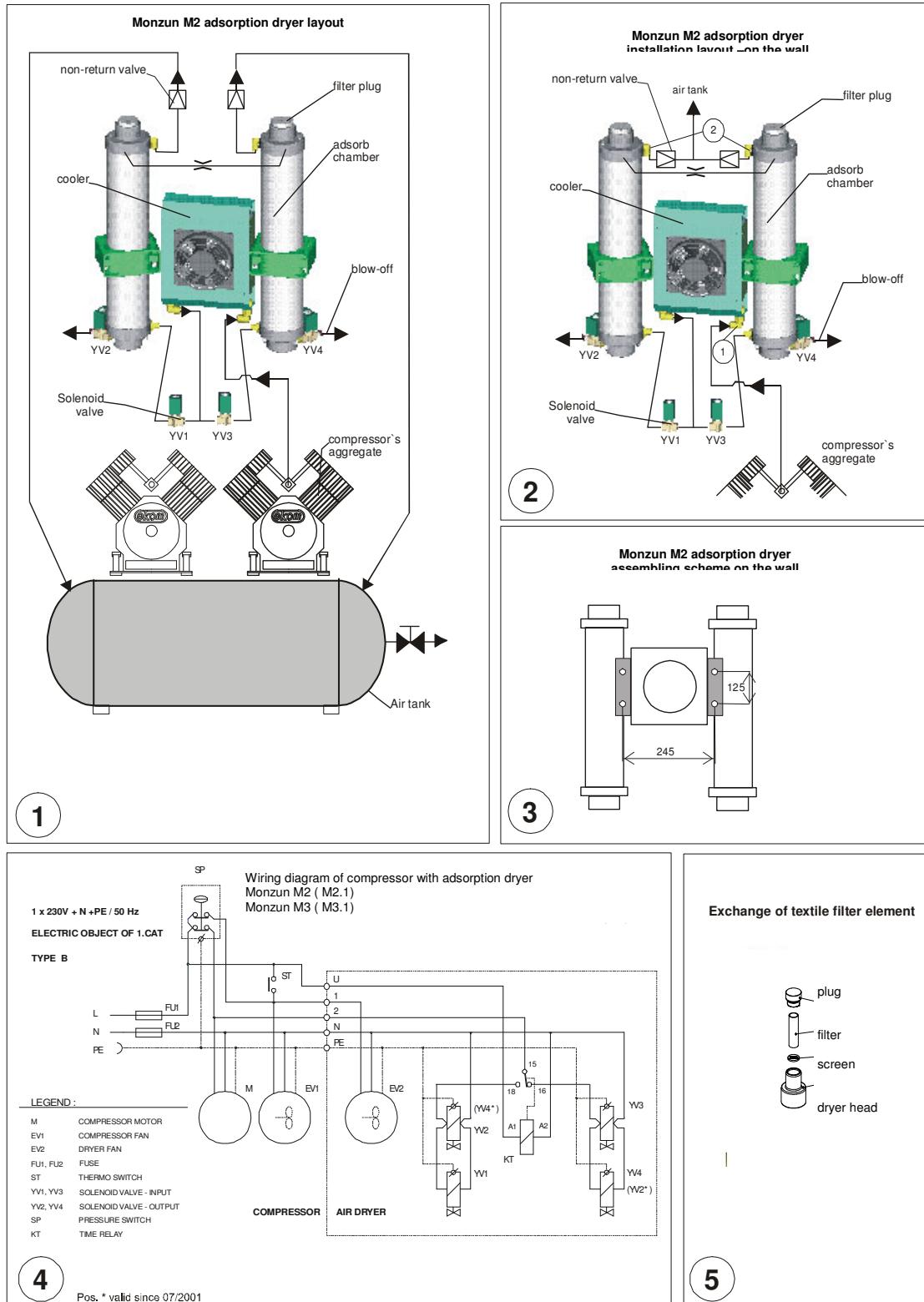
M2, M2.1

Up to 140 l/min (DK50 2V/110)  
Up to 280 l/min (DK50 2x2V/110)

ø0,7mm  
ø1 mm

Manufacturer reserves the right to make changes on the appliance, that however shall not affect substantial properties of the appliance.

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### Certifikat:

Product is manufactured according to production documentation and approved technical conditions. The product is complete and meets all prescribed conditions.

### Garantee:

Manufacturer provides a guarantee of the product in accordance with the paragraphs No.429, 430 and 431 of the Commercial Act No.513/1991, in duration of 24 months from the selling day, when the terms of guarantee document are followed.

- in case of the claim for guarantee reparation a correct fulfilled guarantee document and certificate of quality and completeness of product must be submitted together with the product itself.
- at the guarantee period all failures appeared on the product due to production fault will be removed without any payment.
- the guarantee does not apply to the product damaged during transportation or during incorrect storage.
- the guarantee does not apply to the failures due to wrong operation.
- the guarantee does not apply to the failures due to use of product for any other purposes than are described in the directions for use or as has been understood with the manufacturer.
- the guarantee does not apply to the product when an intervention or arbitrary correction has been made.
- the guarantee does not apply to the product with any completeness which should be discovered during the sale.

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